



TECHNICAL SPECIFICATION

Version 3.0



15 Nov 2013

All information in the document are highly confidential and no part of contents may be informed or transmitted to any party without permission from ALPINION MEDICAL SYSTEMS.

E CUBE 7

SYSTEM SPECIFICATION

Featuring ALPINION's core imaging technologies, the E-CUBE 7 delivers user superior image quality without case dependency.

E³
E cubed

ERGONOMIC DESIGN
EFFICIENT WORKFLOW
EXTREME ACCURACY



01 SYSTEM INTRODUCTION

PHYSICAL DIMENSIONS

- Weight: 57kg
- Height: 1355 (short)/ 1425 (tall) mm (minimum-monitor folded: 1070mm(S), 1140mm (T))
- Width: 509mm
- Depth: 670mm

MONITOR

- 18.5" Wide LED
- Brightness, contrast adjustment (OSD button)
- IPS (In plane switching) technology
- Monitor tilt (Monitor ARM)
 - > 15 degree up, 90 degree down
 - > ±135 degree swivel
- Integrated stereo speakers
- Display size: 1366 X 768
- Recording area: 880 X 660

Monitor ARM

- Adjustable Tilt/Swivel
- Safety space for protecting of hands during monitor folding operation

TRANSDUCER CONNECTOR

- 3 active Transducer ports (Optional High density array port: 3rd port)

CONTROL PANEL

- Back-lit Alphanumeric keyboard
- Alphanumeric QWERTY keyboard
- 8 steps TGC (Time Gain Compensation) keys
- 5 Soft keys
- 2 User define keys
- 14 Power preset keys
- On/Off task light and back-lit illumination of control panel

CONSOLE DESIGN AND USER INTERFACE

- 3 active Transducer ports
- Integrated HDD (Capacity: over 500GB)
- Integrated DVD-R/W Drive
- On-board Storage for Peripherals
 - > B/W Printer, Color Printer or DVD recorder
- Control panel Fixed Type
- 3 Transducer holders, detachable for cleaning and washing
- Air Filters
- Front Handle
- Wheel-lock Mechanism
 - > Front-wheel & Back-wheel: Total Lock
- 6 USB ports: Front side (2 ea), Back side (4 ea)
- Thumbnail images on-screen
- On-line Help key

HARD DISK DRIVE

- Internal 385 GB hard disk drive for patient database management

ELECTRICAL POWER

- Voltage: 100 - 120V, 220 - 240V
- Frequency: 50/60Hz
- Power: Max. 600 VA with Built-in and On-Board Peripherals

SYSTEM ON/OFF AND RESPONSE TIME

- Boot up time: 120 sec
- Shutdown time: 30 sec
- Response time: 0.5 sec (B-mode --> Duplex mode), 1.0 sec (B-mode --> Triplex mode)

LANGUAGE SUPPORT

- English
- German
- French
- Spanish
- Italian
- Russian

- Brazilian

02 SYSTEM OVERVIEW

APPLICATIONS

- Abdominal
- Obstetrical
- Gynecological
- Cardiac
- Vascular
- Urological
- Small Parts and Superficial
- Pediatric and Neonatal
- Transcranial
- Emergency Medicine

OPERATING MODES

- B-mode
- M-mode
- Pulsed Wave (PW) Doppler mode
- Continuous Wave (CW) Doppler mode
- Color Flow (CF) mode
- Power Doppler (CF) mode
- Anatomical M-mode
- THI (Tissue Harmonic Imaging, PI/FTHI)
- Beam Steering
- Directional Power Doppler mode
- SRI (Speckle Reduction Imaging)
- FSRI (Full Speckle Reduction Imaging)
- Spatial compounding
- Frequency compounding
- Auto IMT
- Auto traces PW
- Panoramic B/CF
- Xspeed on 2D / CF/PW
- 3D/4D Volume mode

DISPLAY MODES

- Duplex mode
 - > PW Doppler mode (B/PW)
 - > CW Doppler mode (B/PW)
 - > Color Flow mode (B/CFM)
 - > Power Doppler mode (B/PDI)
 - > Motion mode (B/M)
 - > Directional Power Doppler mode
- Real Time Triplex mode (B /CFM/PW, B/CFM/CW, B/PDI/PW, B/PDI/CW)
- Zoom: Write/ Read/Pan (Write zoom up to 8x)
- Colorized Image (B, M, PW, CW)
- Virtual Convex
 - > Left/Right steer

- > Trapezoid Imaging
- Full Screen
- Quad Screen Display
- Time Line Display
 - > Independent Dual B/PW Display
 - > Display Formats
 - Vertical: 1/2, 1/3, 2/3
 - Horizontal: 1/2, 1/3, 2/3
 - > Full: Time Line Only (PW / M)
- Maximum Depth: 30cm

DISPLAY ANNOTATION

- Institution/Hospital Name: 25 Characters
- Date: 3 types selectable
 - > YYYY/MM/DD, MM/DD/YYYY, DD/MM/YYYY
- Time: 2 types selectable
 - > 24 hours, 12 hours
- Operator Identification
- Patient Name: First, last, middle name
- Patient Identification: 64 Characters
- Gestational Age form
 - > LMP/EDC/GA
- Acoustic power output
 - > MI (Mechanical Index)
 - > TIS (Thermal Index Soft Tissue)
 - > TIC (Thermal Index Cranial (Bone))
 - > TIB (Thermal Index Bone)
- System Status (real-time or frozen)
- Transducer Directional Marker
- Image Preview: Thumbnails
- Gray/Color Bar
- Cine Gauge
- Measurement Summary Window
- Measurement Results Window: pre-settable display location
- Transducer Type
- Application Name
- Imaging Parameters by mode (current mode highlighted)
 - > B mode
 - Imaging Frequency
 - Dynamic Range
 - Rejection
 - Virtual
 - Angle Steer
 - Gray Map
 - Colorize
 - Up/Down
 - Power Output
 - SRI
 - FSRI
 - Persist
 - Spatial Compounding

- Line Density
- Frequency Compounding
- > M mode
 - Anatomical M-mode
 - Dynamic Range
 - Rejection
 - AMM Angle
 - Sweep Speed
 - Gray Map
 - Colorize
 - Full Screen M mode
 - Power Output
- > Color Flow mode
 - Doppler Frequency
 - PRF
 - Angle Steer
 - Baseline
 - Invert
 - Wall Filter
 - Persist
 - Color Map
 - Threshold
 - Power Output
 - Ensemble
 - Line Density
 - Smooth
- > Power Doppler mode
 - Doppler Frequency
 - PRF
 - Angle Steer
 - Wall Filter
 - Persist
 - Color Map
 - Threshold
 - Power Output
 - Ensemble
 - Line Density
 - Smooth
- > PW mode
 - Sample Volume Width
 - PRF
 - Angle Steer
 - Base Line
 - Invert
 - Doppler Frequency
 - Wall Filter
 - Angle Correct
 - Sweep Speed
 - Full Screen PW mode
 - Power Output

- Rejection
- Dynamic Range
- Gray Map
- Colorize
- Time Resolution
- Update
- Auto Calculation
- Method
- Direction
- Sensitivity
- > CW mode
 - PRF
 - Baseline
 - Inverse
 - Doppler Frequency
 - Wall Filter
 - Angle Correct
 - Sweep Speed
 - Full Screen CW mode
 - Power Output
 - Rejection
 - Dynamic Range
 - Gray Map
 - Colorize
 - Time Resolution
 - Auto Calculation
 - Method
 - Direction
 - Sensitivity
- TGC Curve: On/Off
- Body Pattern: 164 types
- B Scale Markers
- M Scale Markers
 - > Time/Depth
- Caps Lock: On/Off
- System Message Display
- Trackball Functionality Status Display
- Heart Rate
- Biopsy Guide Line and Zone
- Focal Zone mark

ANNOTATION PACKAGE

- Arrow
 - > Arrow size: S, M, L, XL
 - > Rotate Arrow
- Body pattern
- Text
 - > Font Color: Green, Yellow, White, Orange
 - > Text size: S, M, L

IMAGE PROCESSING

B MODE

- Gain: 0-90 db (1db increment)
- Imaging Frequency: 3 Selectable Imaging frequencies
- Dynamic Range: Up to 192 dB
- Rejection: 10 steps
- Virtual: ON/OFF
- Angle Steer: 6 steps
- Gray Map: 14 steps (0-13)
- Colorize: 21 steps (0-20)
- Up/Down: ON/OFF
- Power Output: 1-100% (2% increment)
- SRI: ON/OFF
- FSRI: 5 steps
- Persist: 4 steps
- Spatial Compounding: 3 steps
- Line Density: 5 steps
- Frequency Compounding: ON/OFF
- Transmit Focus position: 30 position types
- Multi Focus: Max 8

M MODE

- Anatomical M-mode: On/Off
- Dynamic Range: 30-150 dB
- Rejection: 10 steps
- Sweep Speed: 5 steps
- Gray Map: 14 steps (0-13)
- Colorize: 19 steps (0-18)
- Full Screen M mode: ON/OFF
- Power Output: 1-100% (2% increment)

PW MODE

- SV Gate Width: 13 steps (0.7,1,2,3,4,5,6,7,8,9,10,11,15)
- PRF: 300Hz – 20100Hz (Transducer dependent)
- Angle Steer: 6 steps
- Base Line: 16 steps
- Invert: ON/OFF
- Doppler Frequency: 3 selectable frequencies
- Wall Filter: 9 steps
- Maximum/Minimum Velocity Scales
 - > Max: 52m/sec (Angle/Transducer dependent)
 - > Min: 10cm/sec
- Angle Correct: $\pm 89^\circ$, (1° step)
- Sweep Speed: 5 steps
- Full Screen PW mode: ON/OFF
- Power Output: 1-100% (2% increments)
- Rejection: 10 steps
- Dynamic Range: 30-120dB
- Gray Map: 14 steps (0-13)
- Colorize: 19 steps (0-18)

- Time Resolution: 1-7
- Update: Frozen, Live, 2,3,4,8,16s
- Auto Calculation: ON/OFF
- Method: Mean, Max, Both
- Direction: Below, Above, Both
- Sensitivity: 20 steps

CW MODE

- PRF: 500Hz – 78100Hz (Transducer dependent)
- Baseline: 16 steps
- Inverse: ON/OFF
- Doppler Frequency: 2 selectable frequencies
- Wall Filter: 9 steps
- Maximum/Minimum Velocity Scales
 - > Max: 270m/sec (Angle/Transducer dependent)
 - > Min: 10cm/sec
- Angle Correct: $\pm 89^\circ$
- Sweep Speed: 5 steps
- Full Screen CW mode: ON/OFF
- Power Output: 0~100%
- Rejection: 10 steps
- Dynamic Range: 30-120dB
- Gray Map: 14 steps (0-13)
- Colorize: 19 steps (0-18)
- Time Resolution: 1-7
- Auto Calculation: ON/OFF
- Method: Mean, Max, Both
- Direction: Below, Above, Both
- Sensitivity: 20 steps

COLOR FLOW MODE

- Doppler Frequency: 3 selectable frequencies
- PRF: 300Hz-20,100Hz (Transducer dependent)
- Angle Steer: 6 steps
- Baseline: 40 steps
- Invert: ON/OFF
- Wall Filter: 7 steps
- Persist: 10 steps
- Color Map: 10 steps (0-9)
- Threshold: 0-100%
- Power Output: 1-100%
- Ensemble: 6-16
- Line Density: 2 steps
- Smooth: 10 steps

POWER DOPPLER MODE

- Doppler Frequency: 3 selectable frequencies
- PRF: 300Hz-20,100Hz (Transducer dependent)
- Angle Steer: 6 steps
- Wall Filter: 7 steps

- Persist: 10 steps
- Color Map: 10 steps (0-9)
- Threshold: 0-100%
- Power Output: 1-100%
- Ensemble: 6-16
- Line Density: 2 steps
- Smooth: 10 steps

VOLUME MODE

- Rendering mode
 - > Surface (Gradient/Texture)
 - > Light
 - > MIN IP (Intensity Projection)
 - > MAX IP (Intensity Projection)
 - > X-Ray
- Viewing volume data
 - > 3D/4D (Live and Review)
- MPR + VR
- CUBE CT
- Slices
 - > Multi slice
- Editing volume
 - > Inside & Outside Contour/Box
- Annotating volume data
 - > Comment
 - > Arrow
- Navigation
- Cine
 - > 3D Rotation
 - > 4D Cine Loop

CINELOOP REVIEW

- 3,000 frames CINE memory
- Cine replay speed: 200%, 100%, 50%, 25% (4 types)
- Cine gauge and cine image number display
- Cine review: Frame by frame, Loop
- Start and End Frame Selections for Loop Playback
- Measurement and calculation capability

IMAGE ARCHIVE/CONNECTIVITY

- Preview: displays thumbnail images of the acquired data for the current exam
- E-View: An enlarged preview of the image
- Recalling Images from the Preview
- Image Management
 - > Select All/Unselect All
 - > Permanent Store
- Hard disk drive Image Storage: Min 300GB
- Ethernet Network Connection
- Archiving Format:
 - > Standard DICOM (US/MF)

- > Secondary Capture
- 6 USB ports
- DVD/CD writes and read capabilities
- Export Image Format
 - > Bitmap
 - > JPEG
 - > DICOM
 - > WMV
 - > AVI (Volume Cine)
- DICOM 3.0 Connectivity
 - > DICOM Structured Report
 - > DICOM Verification
 - > DICOM Storage
 - > DICOM Storage Commitment
 - > Modality Worklist
 - > MPPS
 - > Print
- Network Storage

03 MEASUREMENTS/CALCULATIONS

BASIC MEASUREMENTS/CALCULATIONS

B MODE

- Distance
- Ellipse
- Trace
- % Stenosis
- Volume
- Ratio
- Angle
- Histogram

PW MODE

- Velocity
- PI (Pulsatility Index)
- RI (Resistance Index)
- S/D Ratio (Systole/Diastole Ratio)
- A/B Ratio
- PG Mean (Pressure Gradient Mean)
- PG Max (Pressure Gradient Max.)
- Acceleration
- HR (Heart Rate)
- Time (Velocity Time)

M MODE

- Distance
- HR (Heart Rate)
- Slope
- % Stenosis
- Time
- Ratio (% Distance)

B/PW MODE

- Auto & Manual Trace
 - > PS (Peak Systole)
 - > ED (End Diastole)
 - > MD (Minimum Diastole)
 - > PS/ED (Peak Systole/End Diastole)
 - > ED/PS (End Diastole/ Peak Systole)
 - > PI (Pulsatility Index)
 - > RI (Resistance Index)
 - > TAmx (Time avg. max. Velocity)
 - > TAmx (Time avg. mean. Velocity)
 - > VTI (Velocity Time Integral)
 - > HR (Heart Rate)

LABELED MEASUREMENTS/CALCULATIONS

CARDIOLOGY MEASUREMENTS/CALCULATIONS

B mode

- AV/LA (Aortic Valve/Left Atrium): RV, LA & Ao Dm
- PA (Pulmonary artery): PA Dm
- Vena Cava: IVC & SVC Dm
- RV (Right Ventricle): RV Diameter, RV length
- Simpson BP (Simpson Bi-plane): EDV & ESV
- Simpson SP (Simpson Single-plane): EDV&ESV
- Modified Simpson
- Area Length: LVLd, LVLs, LVAd, LVAs
- Teichholz (Left Ventricular Dimensions by Teichholz method):RVAWd, RVDd, Diastole, Systole
- LV Mass (Left Ventricle Mass): Truncated Ellipse & Area-Length method
- LA Vol A-L (Left Atrium Volume by Area-Length method)
- LA Vol /Simp BP (Left Atrium Volume by Simpson method /Biplane)
- RA Vol /A-L (Right Atrium Volume by Area-Length method)
- RA Vol /Simp (Right Atrium Volume by Simpson method /Single)
- MV (Mitral Valve): EPSS, LVOT Dm, MV Area, MV Dm
- AV (Aortic Valve): LVOT Dm, AVA Area
- MR (Mitral Valve - Regurgitant Flow): MR VC Dm, Jet Area
- AR (Aortic Valve - Regurgitant Flow): AR VC Dm, Jet Area
- TR (Tricuspid Valve - Regurgitant Flow): TR VC Dm, RAP
- PV (Pulmonary Valve): PV Dm
- PVe (Pulmonary Vein): PVed Dm, PVes Dm
- PISA AR (Proximal Isovelocity Surface Area of Aortic Regurgitation): Radius, Aliasing Vel
- PISA MR (Proximal Isovelocity Surface Area of Mitral Regurgitation): Radius, Aliasing Vel

M mode

- Teichholz (Left Ventricular Dimensions by Teichholz method): RVAWd, RVDd, Diastole, Systole, LVET, HR
- AV/LA (Aortic Valve/Left Atrium): RV, LA & Ao Dm LVET, LVPEP
- MV (Mitral Valve): CA/CE amp, DE amp/slope, EPSS, EF slope,
- RV (Right Ventricle): RV Dm, RVOT Dm
- PVe (Pulmonary Vein): PVe Dm

Doppler mode

- MV (Mitral Valve): E Dur, A Dur, IVRT, MV E pt, MV A pt, MVA (PHT, VTI, Area), CO, LVIMP, HR
- AV (Aortic Valve): AV VTI, LVOT VTI, AVA (Vmax, Area)
- PV (Pulmonary Valve): PV Vmax, CO
- TV (Tricuspid Valve): TV VTI, TV Vmax, TV E pt, TV A pt, RVIMP
- PVe (Pulmonary Vein): PVs, PVd, PVa
- AR (Aortic Valve - Regurgitant Flow): AI Decel slope, AI PHT, AR VTI
- TR (Tricuspid Valve - Regurgitant Flow): TR VTI, TR VC Dm, RAP
- PR (Pulmonary Valve - Regurgitant Flow): PR VTI, PR V ed
- MR (Mitral Valve - Regurgitant Flow): MR Vmax, dp/dt, MR VC Dm
- PISA AR (Proximal Isovelocity Surface Area of Aortic Regurgitation): AR VTI, Aliasing Vel
- PISA MR (Proximal Isovelocity Surface Area of Mitral Regurgitation): MR VTI, Aliasing Vel
- TDI (Tissue Doppler Imaging): MV E pt, Ea, Aa, Sa

OBSTETRICS MEASUREMENTS/CALCULATIONS

- Abdominal Circumference (AC)
- Anterior Posterior Thoracic Diameter (APTD)
- Binocular Distance (BOD)
- Biparietal Diameter (BPD)
- Clavicle (CLAV)
- Crown Rump Length (CRL)
- Estimated Fetal Weight (EFW)
- Fibula (FIB)
- Femur Length (FL)
- Fetal Trunk Area (FTA)
- Gestational Sac (GS)
- Head Circumference (HC)
- Humerus
- Middle Abdomen Diameter (MAD)
- Occipital Frontal Diameter (OFD)
- Radius
- Spinal Length (SL)
- Transverse Abdominal Diameter (TAD)
- Transverse Cerebella Diameter (TCD)
- Tibia
- Transverse Thoracic Diameter (TTD)
- Ulna Length (ULNA)
- Multi-Gestational Calculation
 - > Up to 4 fetuses comparison of multiple fetuses data on a graph and a worksheet
- OB Worksheet
- Patient Information
 - > Fetus Number
 - > CUA/AUA Selection
 - > Fetus Position
- Measurement Information

REPORT PACKAGE

- Abdomen
- Obstetrics
- Gynecology
- Cardiology
- Vascular
- Urology
- Pediatrics
- Small Parts
- Breast
- MSK
- EM (Emergency Medicine)

TRANSDUCER SPECIFICATION

SC1-6*

- Applications: Abdomen, Renal, OB, Fetal Echo, GYN, Emergency Medicine
- Transducer Type: Convex array (Premium*)
- Frequency Bandwidth: 1.0 - 6.0 MHz
- Convex Radius (mm): 60 mm
- FOV: 60°
- Number of element: 128
- Biopsy kit: Available

*Note: Premium means single crystal as piezoelectric material.

C1-6

- Applications: Abdomen, Renal, OB, Fetal Echo, GYN, Emergency Medicine
- Transducer Type: Convex array
- Frequency Bandwidth: 1.0 - 6.0 MHz
- Convex Radius (mm): 60 mm
- FOV: 60°
- Number of element: 128
- Biopsy kit: Available

C5-8

- Applications: Abdomen, Cardiac, Emergency Medicine
- Transducer Type: Micro Convex array
- Frequency Bandwidth: 5.0 - 8.0 MHz
- Convex Radius (mm): 14 mm
- FOV: 92°
- Number of element: 128
- Biopsy kit: N/A

L3-12H*

- Applications: Carotid, Peripheral Vascular, Thyroid, Testicle, MSK, Superficial, Breast, Emergency Medicine
- Transducer Type: Linear array
- Frequency Bandwidth: 3.0 - 12.0 MHz
- Foot Print: 45 mm
- Biopsy kit: Available
- Number of element: 192

Note: H means high density transducer.

L3-12H^{WD}*

- Applications: Carotid, Peripheral Vascular, Thyroid, Testicle, MSK, Superficial, Breast, Emergency Medicine
- Transducer Type: Linear array
- Frequency Bandwidth: 3.0 - 12.0 MHz
- Aperture length (mm): 64 mm
- Number of element: 192
- Biopsy kit: N/A

Note: H means high density transducer.

Note: WD means wide foot print transducer.

L3-12

- Applications: Carotid, Peripheral Vascular, Thyroid, Testicle, MSK, Superficial, Breast, Emergency Medicine
- Transducer Type: Linear array
- Frequency Bandwidth: 3.0 - 12.0 MHz
- Foot Print: 45 mm
- Number of element: 128
- Biopsy kit: Available

L3-8

- Applications: Carotid, Peripheral Vascular, Thyroid, Testicle, MSK, Superficial, Breast, Emergency Medicine
- Transducer Type: Linear array
- Frequency Bandwidth: 3.0 - 8.0 MHz
- Foot Print: 45 mm
- Number of element: 128
- Biopsy kit: Available

L8-17

- Applications: Carotid, Peripheral Vascular, Thyroid, Testicle, MSK, Superficial, Breast, Emergency Medicine
- Transducer Type: Linear array
- Frequency Bandwidth: 8.0 - 17.0 MHz
- Foot Print: 32 mm
- Number of element: 128
- Biopsy kit: Available

SP1-5

- Applications: Abdomen, Renal, Cardiac, Emergency Medicine, TCD
- Transducer Type: Phased array (Premium*)
- Frequency Bandwidth: 1.0 - 5.0 MHz
- FOV: 90°
- Number of element: 64
- Biopsy kit: N/A

* Note: Premium means single crystal as piezoelectric material.

SP3-8

- Applications: Abdomen, Renal, Cardiac, Emergency Medicine, TCD
- Transducer Type: Phased array (Premium*)
- Frequency Bandwidth: 3.0 – 8.0 MHz
- FOV: 90°
- Number of element: 64
- Biopsy kit: N/A

VC1-6

- Applications: Abdomen, Renal, OB, Fetal Echo, GYN, Emergency Medicine
- Transducer Type: Volume Convex array
- Frequency Bandwidth: 1.0 - 6.0 MHz
- Convex Radius (mm): 40 mm
- FOV: 79°
- Number of element: 128
- Biopsy kit: N/A

EN3-10

- Applications: OB, Fetal Echo, GYN, Urology, Emergency Medicine
- Transducer Type: Endovaginal/Endocavity
- Frequency Bandwidth: 3.0 - 10.0 MHz
- Convex Radius (mm): 10 mm
- FOV: 145°
- Number of element: 128
- Biopsy kit: Available

E3-10

- Applications: OB, Fetal Echo, GYN, Urology, Emergency Medicine
- Transducer Type: Endovaginal/Endocavity
- Frequency Bandwidth: 3.0 - 10.0 MHz
- Convex Radius (mm): 10 mm
- FOV: 145°
- Number of element: 128
- Biopsy kit: Available

ALPINION Medical Systems

Tel: 82.70.7465.2000

Fax: 82.2.851.5595

Email: ecube@alpinion.com