



THERATOOGS™

Wunzi System

Basic Care Instructions

Worn Daily

Wash in WATER only every other day and gentle cycle machine wash with hydrogen peroxide once a week.

Not Worn Daily

Wash in WATER only every three days and gentle cycle machine wash with hydrogen peroxide once a week.

Please see the Product Care Instructions for further details.

TO START:

Lay your infant face up on the foam side of the large Back Panel. Then...


3 Steps to Dress Your Child

1



1. Attach shoulder straps.

2

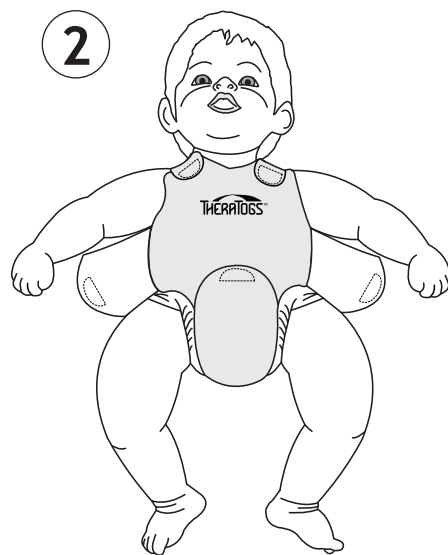


2. Attach diaper flap.

3



3. Attach rib flaps.

EXTRA: To Promote Back Extension

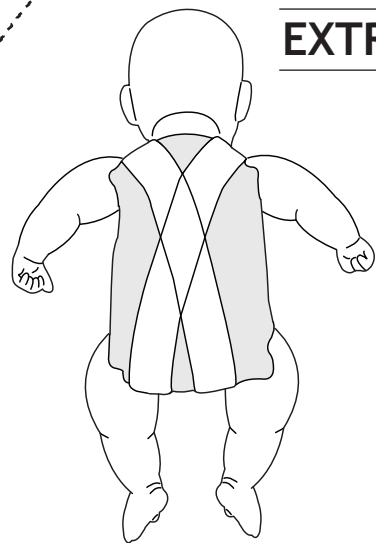

1. Position Cross Strap lengthwise down the back and attach top ends to shoulders with the white 'hook' tabs.
2. Stretch Cross Strap gently down to create tension.
3. Attach bottom ends with tabs under the diaper at the proper point to maintain desired tension.
4. Cut off any excess strapping length.

TheraTogs are patented FDA Class I medical devices intended to be issued by, and applied under the supervision of, a licensed healthcare practitioner engaged in neuromotor or sensorimotor retraining.

TheraTogs GoldTone™ fabric and TogRite™ strapping are latex-free.

TheraTogs™, GoldTone™ and Wunzi™ are trademarks of TheraTogs, Inc. US Patent #8007457B2; additional US and foreign patents pending. All Rights Reserved.

NEED HELP? CONTACT CLINICAL SUPPORT

888-634-0495
(01) 970-239-0108
support@theratogs.com
www.theratogs.com/wunzi

TheraTogs are not considered sleepwear and are not flame resistant.